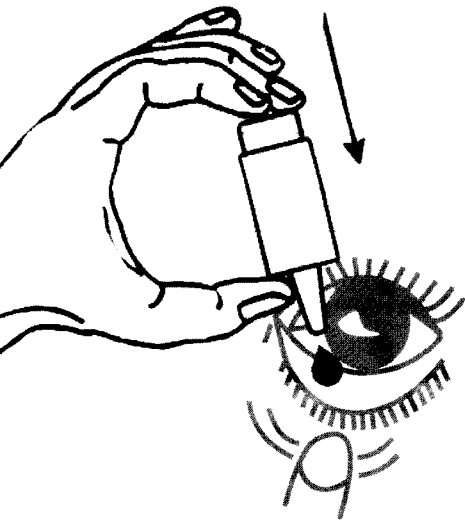
# DrugStore.co.il

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### HYLO-COMOD®

#### Sterile solution preservative free

- Composition:** 1 ml solution contains 1 mg sodium hyaluronate, monobasic sodium phosphate, dibasic sodium phosphate, glucitol and water for injection.
- Indication:** Solution for rinsing and wetting contact lenses inside and outside the eye. For improving the hydration of the surface of the eye.
- Contraindication:** Do not use if a known sensitivity exists to any of the ingredients.
- Interactions:** HYLO-COMOD® should not be used concomitantly with ophthalmic drugs. If you have to apply eye drops please wait for about 30 minutes between the two applications. Eye ointments should be used after the application of HYLO-COMOD® in any case.
- Dosage:** Can be used as needed or as instructed by your optician.

If using HYLO-COMOD® for the first time the container has to be primed by pressing the base several times until a metered drop is expelled. HYLO-COMOD® is now ready for use. Each pressing on the base of the bottle will produce a metered dose, As it is shown in the picture.

Because of the special construction of valves the out coming drops are always of equal size and speed not depending on how powerfully you press on the bottom.

Tilt your head slightly back, retract the lower lid of the eye and apply one drop as described into the conjunctival sac. Close the eye slowly so that the solution can be spread on the surface of the eye. After use close the bottle carefully with the blue cap. Avoid contact of the bottle tip with eye or skin. You can take 10 ml of the eye drops out of one bottle of HYLO-COMOD® so that usually a bottle lasts up to 3 months.

- Storage:** Store at room temperature (15-25°C). Store out of the reach of children. Not to be used longer than 12 weeks after the bottle has been opened.

- Package:** 10 ml sterile bottle usually lasts for up to 3 months.

**Manufacturer:** URSAPHARM Arzneimittel GmbH & Co. KG  
**Distributor:** Kivema Ltd P.O.Box 33 Hachosh St, Kfar Shmaryahu.

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KIVEMA LTD